

BrailleNote Touch Plus Release Notes

Customize your menu

- Press Backspace + Enter + X to access your custom menu.
- Remove preloaded apps by pressing Space + X or add apps from the All applications menu by pressing Enter + M.

Copy last utterance

- Copy anything that is spoken including from non-editable fields.
- MUST have the speech on to copy.
- Enter + S will start the selection, pressing Enter + S again will stop the selection.
- Use general KeySoft shortcuts to copy and paste.

Easy Reader +

- Added additional language support including Arabic, Hebrew and Spanish.
- Added the Éole library from Association Valentin Haüy in France.

This system update also includes improvements to the Android platform to follow Google Certification regulations.