explorē 8: Update v1.3.0

We are delighted to introduce a new, optional, foldable reading stand for explorē 8 handheld magnifiers. Multiple new features, usability improvements, and fixes accompany the stand with software update version 1.3.0.

Features

All explorē 8 Devices

Red Color Schemes: New text and color contrast combinations are now available to accommodate specific user needs: red text on a white background, white on red, black on red, and red on black.

Auto Shutoff: A “Never” option has been added to the existing auto shutoff choices of 1, 2, 5, and 10 minutes, preventing the device from shutting off during use.

Lights On/Off: In handheld/distance mode, users can select whether the lights are on or off, conserving battery life and optimizing usability.

For Use with the Reading Stand

Preferred Zoom Setting: When using the stand, users can choose a preferred zoom level between 2X and 15X, saving time and enhancing convenience.

Lights On/Off: In stand mode, users can select whether the lights are on or off, conserving battery life and optimizing usability.

Temporary Auto-Focus Gesture: Users can double-tap the screen with a finger to temporarily lock the focus while using the device in the stand. This feature prevents the device from refocusing, allowing tasks like document signing without interruption.

Overview Gesture: A new gesture, activated by placing two fingers at the top of the screen and drawing them down, zooms the device to 1X for better orientation. Removing fingers returns to the previous zoom level.

Text/Handheld-Distance Toggle: A new touchscreen button lets users quickly switch between stand reading and handheld distance viewing modes.

Important Note

As part of the latest update, we've changed the explorē 8's storage system to enhance overall performance. To implement these changes, all existing gallery files will be deleted during the update process. Please back up any images you wish to keep before proceeding with the update.

To learn more, visit the [explorē 8 stand](#) page of our website.

Visit www.humanware.com/exploresupport to access the v1.3.0 software update and available resources.

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