




Annie Knowledge Levels

How to update Annie


There are multiple ways to update your Annie. If an update is available and you are connected to the Internet Annie will prompt you that an update is readily available. In the event you are unable to use the internet to upgrade you can also download the update file from the HumanWare support page.

To start the update follow this procedure:

1. From the main menu, navigate to the settings item, then press space to select.

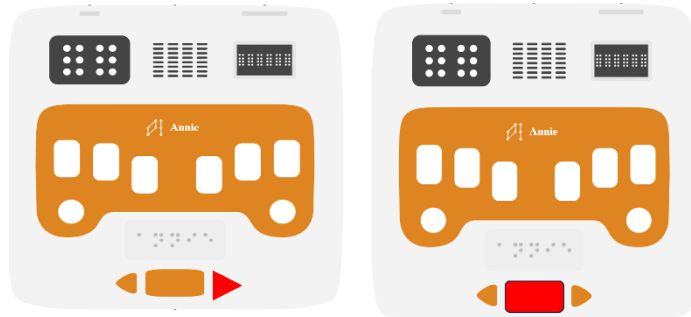

2. Using your arrow navigation keys, navigate to the Check for updates item, then press space to select.

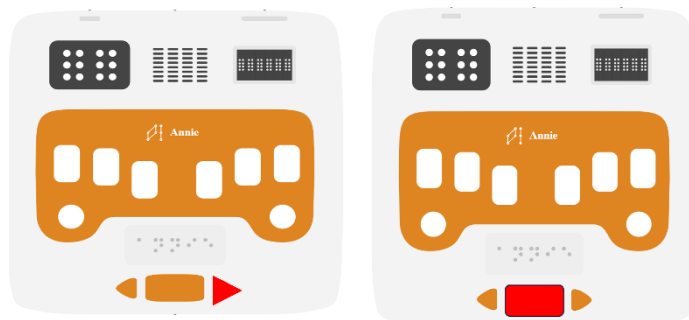

3. You will now have two options available. These options are:
 - a. Update using USB Drive
 - b. Update over the Internet
4. Using your arrow navigation keys move to the desired option and press Space to select.

